

# CaraClip™


wire protector

Sterile, adhesive clips to secure and hold guidewires, catheters, sheaths/tubing or other accessories during interventional, surgical, and other procedures.

## Instructions For Use:

1. Remove liner backing to expose adhesive.
2. Apply firmly to ensure adhesion.
3. To insert or remove device(s) into/from lumen of CaraClip:
  - a. press down on the long arm of the clip, or
  - b. advance through lumen

**Note: NOT FOR PINNING**

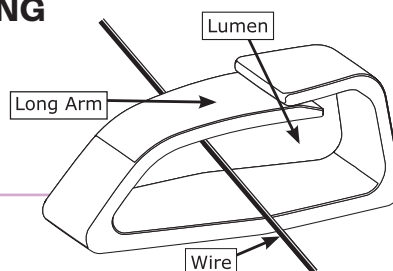


## Distributed By:



### Medical Resources Group

1920 South Main Street, Morton, IL 61550

+1-309-263-6901 | [www.mrg-ltd.com](http://www.mrg-ltd.com) | [sales@mrg-ltd.com](mailto:sales@mrg-ltd.com)




 Manufactured by Beekley Corporation  
One Prestige Lane, Bristol, CT 06010-7454 USA  
1.800.233.5539 or +1.860.583.4700 [beekley.com](http://beekley.com)

CARA CLIP is an unregistered trademark of Beekley Corporation.  
® – Reg. U.S. Pat. & Tm. Off.  
© 2023 Beekley Corporation. All rights reserved.  
Pat. – [beekley.com/patents](http://beekley.com/patents)

STERILE R

REF 431

#### WARNINGS AND PRECAUTIONS:

- Do not use if product sterile packaging is compromised. 
- Single patient use only; Do not reuse. 
- Do not resterilize. 
- Does not contain natural rubber latex.

**Beekley Medical®**  
when your diagnosis  
must be right®

REV: CC\_IFU\_1023