


 EN



 | [adeptmedical.com](https://www.adeptmedical.com)

ADEPT

Medical
Supporting you

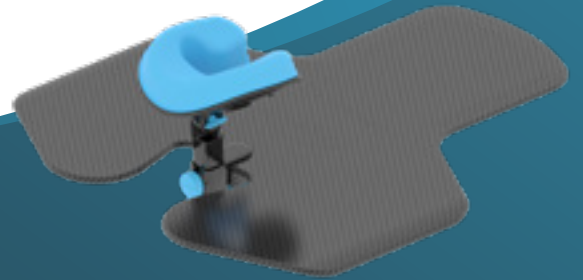


Issue Date: 05/2022

PRONE SUPPORT

For enhanced patient management and comfort.

PRONE SUPPORT

For enhanced patient management and comfort.

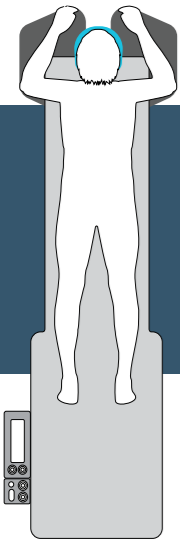

The ideal solution for supporting and managing a prone patient, it provides comfortable patient positioning during image-guided procedures.



This versatile device will support a patient in the prone position for multiple procedure types.




Patient Positioning

The two-part support provides optimal patient comfort and procedural management, supporting both the arms and face.

The arm supporting surface is cantilevered out from the imaging table, accommodating a variety of patient sizes and mobility ranges comfortably. The arms can be positioned according to individual patient shoulder mobility or procedural needs.



The elevated cushioned face support provides additional comfort with ample space beneath for conscious patients. The head supporting assembly can also be removed completely to suit specific patient or case needs.

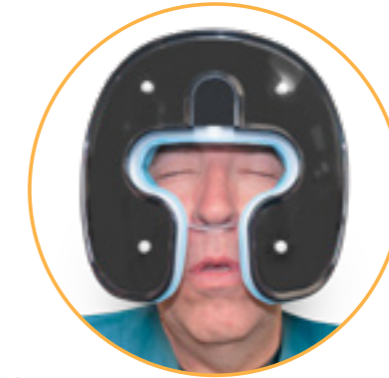

Placement and Adjustment

The Prone Support can be simply placed underneath the mattress using the baseboard design, allowing for efficient set-up and removal on any imaging table. It is stably held for the duration of procedure by the patient's weight. The support design is compatible with most C-Arm imaging centres and tables.



The face support can be height and angular adjusted using the lever lock and thumbscrew, allowing suitability for a range of patient and procedural requirements.



Key Features and Benefits

- | Radiolucent
- | Height and angle adjustment of head support
- | Cushioned face support
- | Versatile arm positions
- | Suits range of patient sizes
- | Quick placement and set-up
- | Suits C-arm tables

Specifications



78cm



*The height adjustable head support offers 85mm of vertical range.

 2.3kg Product Weight

Applications

- Percutaneous Nephrostomy Insertion
- Renal and lung biopsy
- Vertebroplasty / Kyphoplasty
- Percutaneous embolization of sciatic nerve
- Myelogram
- Discogram



Fields you can use this device



Interventional
Radiology



EC Representative

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Manufacturer

Adept Medical Ltd
2-6 McDonald St,
Morningside
Auckland 1025, New Zealand