




Instrument Tray Corner Protectors

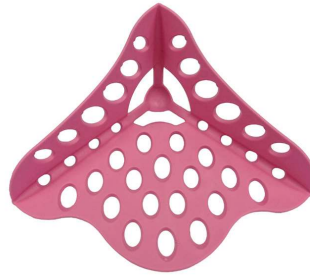

| | |
|--------------------------|---|
| Brand Name of Product | SteriProtect Tray Corner Protector |
| Product Code Number | MRG0400CP / AMO200PC |
| Purpose of Product | To protect the corner of sterilization wrap from sharp edges of sterilization trays |
| Range of Applications | Wrapped trays that are steam, and low temperature plasma sterilization |
| Specification of Product | Thermoplastic Elastomers (TPE). Latex Free. BPA Free |

| Instructions For Use | |
|----------------------|---|
| Description for Use | Place under corner of sterilization tray. Be sure all four corners are covered. |
| Diagram | |

| Reprocessing Instructions | |
|---------------------------|--|
| Manual Cleaning | Instrument Tray Corner Protectors may be manually cleaned with a neutral pH enzymatic detergent and rinsed with running water. You may use a soft bristle brush to remove any contaminants. Do not use scouring pads or abrasive cleaners. |
| Automated Washer | Instrument Tray Corner Protectors may be processed in an automated instrument washer. |